
INSTRUCTIONS FOR USE

CAITHEC PROPHYLAXIS PASTE

1. PRODUCT DESCRIPTION

CAITHEC Prophylaxis Paste is an abrasive dental paste with antiseptic and whitening action that can be used in dental prophylaxis, as it does not interfere with acid etching.

2. PRESENTATION

Tube containing 90 g.

3. COMPOSITION

Calcium carbonate, quartz, sodium sulfate, silica, thickener, sweetener, preservative, flavoring, purified water, glycerin, sodium fluoride.

4. INDICATION OF USE

CAITHEC Prophylaxis Paste is indicated for prophylaxis procedures, helping to remove stains, deposits and bacterial plaque from the dental surface.

5. ACTION

CAITHEC Prophylaxis Paste is classified as an invasive medical product applicable to the body's orifice (oral environment) for transitory use.

6. CONSERVATION/STORAGE

- Keep the product in its original packaging always closed.
- Protect it from direct sunlight.
- Store the product at temperatures between 5°C and 28°C.

7. INSTRUCTIONS FOR USE

- Apply a small amount of CAITHEC Prophylaxis Paste with the rubber cup on the surface of the teeth, using low rotation until complete removal of stains and plaques.
- Then rinse.

8. CONTRAINDICATIONS

- Use low rotation, with little pressure on the felt discs, intermittent touches and in circular movements, thus avoiding excessive heating of the dental surface.
- Do not use it on patients with any sensitivity to the components of this product.

- Suspend the use of the product in case of allergic reactions.

9. WARNINGS

- Restricted use of professionals specialized in dentistry, according to local legislation.
- Strictly follow the procedures provided in the Instructions for Use.
- Care must be taken during the procedure to avoid excessive abrasion of any restorative material present, as these are susceptible to wear due to their low hardness.
- Carefully apply it to areas with exposed dentin.
- Daily use of the product can cause enamel wear.
- Its use in children under 2 years of age may lead to dental fluorosis due to accidental ingestion of this product.
- Do not use the product after the expiry date.
- Keep out of reach of children.

10. SPECIAL CARE

- The dentist must wear protective gloves and goggles when applying the product.

11. MANIPULATION

- Aseptic environment.

12. INTERACTIONS

- No interactions.

