
INSTRUCTIONS FOR USE

CAVITEC
TEMPORARY FILLING PASTE

1. PRODUCT DESCRIPTION

CAVITEC Temporary Filling Paste is a product used for temporary isolation of cavities in restorative and endodontic treatments. When in contact with the patient's saliva, the paste hardens and seals the cavity, preventing contact of the site with the oral environment. However, it is easy to remove when necessary.

2. PRESENTATION

Jar with 25 g.

3. COMPOSITION

Zinc oxide, calcium sulfate and silicone.

4. INDICATION OF USE

This product is indicated for temporary isolation of cavities in restorative and endodontic treatments.

5. ACTION

CAVITEC Temporary Filling Paste has adhesion to the cavity walls and a satisfactory marginal sealing, being extremely important during endodontic procedures, especially when using a delay dressing and after root canal fillings. It prevents microleakage by organic fluids and bacteria from the oral environment and also the passage of intracanal drugs from inside the pulp chamber to the oral environment.

6. CONSERVATION/STORAGE

- Keep the product in its original packaging always closed.
- Protect it from direct sunlight.
- Store it in a dry place at room temperature (5°C to 28°C).

7. INSTRUCTIONS FOR USE

- Use a spatula to apply the desired amount of the product on the site to be temporarily restored.

- The site must be clean and dry.
- Setting time: 3-5 minutes and curing time: 15-30 minutes.

8. CONTRAINDICATIONS

- Do not allow the product to come into contact with the eyes and skin.
- If it comes into contact with the skin or eyes, wash with plenty of water and do not rub (see a doctor immediately).
- Product for professional use only, keep away from those who do not have adequate training.

9. WARNINGS

- Product exclusively for professional use.
- Does not contain eugenol.
- Do not use as a definitive restorative cement.
- Easy removal.

10. SPECIAL CARE

- Keep away from children.

11. MANIPULATION

- The dentist must wear protective gloves and goggles when applying the product.

12. INTERACTIONS

- No interactions.

