

---

## INSTRUCTIONS FOR USE

---

### CAITHEC **FLUORIDE GEL** Neutral

#### 1. PRODUCT DESCRIPTION

CAITHEC Fluoride Gel is used after dental asepsis performed in the office. Its main function is to release fluoride, helping in the process of remineralization of the dental tissue.

#### 2. PRESENTATION

Bottle containing 200 mL.

#### 3. COMPOSITION

Sodium fluoride, carbomer, bi-distilled glycerin, sodium saccharin, propylene glycol, flavor, blue dye, purified water.

#### 4. INDICATION OF USE

Indicated for the prevention of cavities, CAITHEC Fluoride Gel is neutral and for quick application. In addition to being safe, its application is very practical. Its level of viscosity keeps the gel in the WHITEC tray without dripping. Under pressure, its viscosity decreases, and fluoride penetrates into the interproximal areas.

#### 5. ACTION

This product works by slowing enamel erosion and accelerating the natural process of remineralization filling in the weakened areas of enamel.

#### 6. CONSERVATION/STORAGE

- Keep the product in its original packaging always closed.
- Protect it from direct sunlight.
- Store the product at temperatures between 5°C and 28°C.

#### 7. INSTRUCTIONS FOR USE

- After cleaning, dry the teeth well and apply the gel using a tray.
- Leave the WHITEC tray in contact with the dental surface for at least 40 seconds, preventing the patient from swallowing the product (use the sucker if necessary).
- Remove the WHITEC tray and the excess gel and ask the patient to spit out the residue left in the mouth.

- Instruct the patient not to eat or drink anything for at least 30 minutes after application.

#### 8. CONTRAINDICATIONS

- Avoid intentional and excessive ingestion of the product.
- Do not use it on patients with any sensitivity to the components of this product.
- If any problem occurs during or after application, notify your dentist.
- Always keep it away from heat and/or sunlight.
- If a large volume of product is accidentally swallowed, seek medical attention.
- Non-injectable product.
- Do not reuse packaging.

#### 9. WARNINGS

- Ingestion of large amounts of the product may cause nausea.
- Read the instructions for use before using the product.
- For professional use only.

#### 10. SPECIAL CARE

- Suspend the use of the product in case of allergic reactions.
- This is a product composed of fluoride; thus, it should not be ingested.

#### 11. MANIPULATION

- Aseptic environment.

#### 12. INTERACTIONS

- No interactions.

